

# BIOPAD™

## Caring for Wounds

### Preparation and Application

#### ■ Preparation:

1. Prepare the wound bed per the facilities wound care protocol and debride the wound when necessary.
2. Hydrate BioPad with sterile saline if the wound has limited exudate. (BioPad will turn into a gel consistency)
3. If the wound has heavy exudate, rinse out the wound bed with saline prior to application. (Do not hydrate BioPad prior to application)

#### ■ Application:

1. Cut BioPad, if necessary, to fit the size of the wound.
2. Apply directly to the wound bed, covering the entire wound; do not overlap the edges of the wound.
3. Apply a secondary dressing per physician protocol to cover the BioPad.

#### ■ Reapplication:

1. Depending on the amount of exudate, BioPad can be reapplied every day.
2. If the wound has a small amount of exudate, BioPad can be reapplied every 48 hours.
3. BioPad will absorb into the wound bed over time.

#### ■ Indications:

BioPad is indicated for all chronic non-healing wounds that are free of necrotic tissue.

#### ■ Contraindications:

Do not use on patients with a known hypersensitivity to collagen.



#### CONTACT US

**Angelini Pharma Inc.**

8322 Helgerman Court - Gaithersburg - MD 20877 -USA

1 (800) 726-2308

[www.Angelini-us.com](http://www.Angelini-us.com)

**Size:** 2" x 2" - 3 per box

**Order code:** 132622

**HCPCS code:** A6021

20-013-01 Rev. 00 12/13